

ADDENDUM FOR CLINICAL SAMPLES

When working with clinical samples or very hard to pipet samples, we recommend changing the protocol step 1.

DNA purification

1. Spin a 96 Deep Well Plate 1 min at 1000 g and remove seal. Add 0.5 ml milk sample to each of the wells. Cover wells with sealing tape.

Change to

DNA purification

1. Spin a 96 Deep Well Plate 1 min at 1000 g and remove seal. Add 400 µl 10 mM TRIS-HCL buffer pH = 8.0 to each of the wells and then add 100 µl milk sample to each of the wells. Cover wells with sealing tape.

PROTOCOL

For clinical samples
DNA Diagnostic A/S
www.dna-diagnostic.com
Revision 2020.12.03