

COVID-19

Antigen detection (CE/IVD-marked)



Simple
Fast
Cost effective
Accurate
Comfortable
Frequent



COVID-19
antigen detection

- Ensure safety and health
- Keep companies and productions up and running
- Give people peace of mind in difficult times
- Act responsible and be an example to follow
- Comfortable testing. Low risk of damaging glands
- Easy handling. Robust and reliable test

Results within 15 minutes

Our new CV19 antigen detection test kit for screening of coronavirus gives you the result within 15 minutes. Onsite. Furthermore, it's inexpensive and easy to use. It's easy to store and requires no specialist equipment or machinery - making it the perfect choice, if you can't wait for days for the results and if you need to test often to stay ahead of the virus.

The CV19 detection test is CE/IVD-marked.

If safety is the ambition, frequency is the answer

We believe that frequent testing is the key to keeping COVID-19 in check. Our antigen-based COVID-19 tests are nearly as precise as the common known PCR-tests (coincidence rate: 97,3%), but with frequent testing you level out the difference. Instead of relying on day old tests, **you get the results of today.**

Easy to use

The CV19 detection test kit is simple and intuitive. To ensure a good sample and a reliable result, the sample must be taken by a health care professional with a swab inserted into both nostrils of the person. You get a robust test and reliable result within 15 minutes compared to the up to 72 hours of a PCR test.

From events to facility services

Whether you're interested in testing many people once (at e.g. a concert or soccer match), or you need to test a smaller or larger number of people frequently (at e.g. a production facility), we have a solution for you.

Workflow in brief




STEP 1

Dispense all of the buffer into the extraction tube.




STEP 2

Perform a nasal swab. The swab is inserted only 2 cm into the nostril.

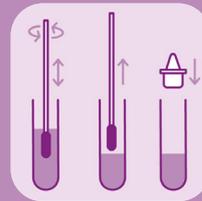


STEP 3

Insert swab into the extraction tube. Close the tube with the drip lid.

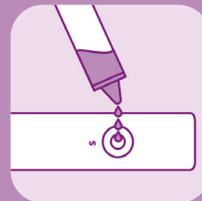

STEP 4

Add 3 drops onto the Detection Card.

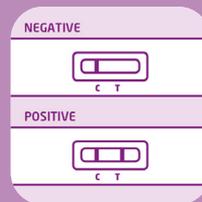

STEP 5

Wait 15 minutes for the result. One band means negative. Two bands mean positive.

For more info visit www.dna-diagnostic.com or contact Tel +45 87 32 30 50 · info@dna-diagnostic.com

**DNA HUMAN
DIAGNOSTIC**

DNA DIAGNOSTIC A/S · VOLDBJERGVEJ 14 · 8240 RISSKOV · DENMARK

About us DNA Human Diagnostic is 100% owned by DNA Diagnostic A/S. DNA Diagnostic A/S is a Danish biotech company founded in 1992, specialized in the development of test kits for Human Health, Food Safety and Animal Health. Our products are sold and used all over the world. At DNA Diagnostic, we believe that **the world deserves fast, cost effective and accurate diagnostics.**

09.02.2021